

Whitening instructions

1. Brush teeth. Remove the syringe cap and insert an application tip by twisting it securely onto the syringe.
2. Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
3. Seat the tray, with the gel around the teeth.
4. Wipe away excess gel in mouth with a tissue or dry soft brush.
5. After treatment, remove tray. Rinse tray and mouth with lukewarm water.
6. Brush teeth.

NOTE:

Do not eat, drink or smoke during treatment. Do not smoke immediately after treatment – wait for at least two hours. Foods and drinks containing strong colours should be avoided for at least 48hours or consumed in moderation. Use gel at room temperature.

Storage and shelf life:

Store in cool place (2°C-25°C/ 35°F -77°F) Do not use after expiration date.

Precautions:

To be used under the supervision of a dentist. Keep out of reach of children under 14 years old. Patients with a history of chemical allergies are advised to carry out allergy testing by a specialist before using these products. Discontinue use if any unusual sensitivity or reactions occur and consult your dentist. DO NOT inject Pola Night into the body. CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR THE ORDER OF THE DENTIST. MSDS available at www.sdi.com.au or contact your regional representative.

FIRST AID:

EYE (contact): Wash thoroughly with water, if symptoms persist seek medical advice.

SKIN (contact): Wash thoroughly with water and seek medical advice if symptoms persist.

INGESTION (large amount): Drink plenty of milk or water and seek medical advice.

Warning!!! Causes skin & eye irritation